

TO DETERMINE ROD LENGTH

MEASURE THE DISTANCE IN INCHES FROM THE FLOOR TO THE ACTUAL MOUNTING SURFACE POINT. MEASURE FROM FLOOR UP TO WHATEVER ACTUAL MOUNTING SURFACE THE LIGHT IS TO BE MOUNTED.

MEDI-LITE & MEDI-SPOT (SOLID MOUNT ONLY)

IF <u>MOUNTING HEIGHT</u> IS:	CEILING <u>ROD LENGTH</u> WILL BE:
8' - 8 1/2'	15" Long
9' - 9 1/2'	25" Long
10' Approx.	35" Long
11' Approx.	45" Long
12' Approx.	55" Long

NOTE: FLEXIGLIDE CEILING TRACKS SHOULD NOT BE MOUNTED HIGHER THAN 9 1/2' AND MUST BE MOUNTED AT THE FINISHED CEILING LEVEL - WHETHER SOLID CEILING OR "DROPPED" CEILING.

ILLUMINATOR IV SERIES (SOLID MOUNT ONLY)

IF <u>MOUNTING HEIGHT</u> IS:	CEILING <u>ROD LENGTH</u> WILL BE:
8' Approx.	18" Long
9' Approx.	30" Long

21ST CENTURY SERIES (SOLID MOUNT ONLY)

IF <u>MOUNTING HEIGHT</u> IS:	CEILING <u>ROD LENGTH</u> WILL BE:
8' Approx.	16" Long
9' Approx.	25" Long
10' Approx.	35" Long

NOTE: SHUTTLETRAK CEILING TRACKS SHOULD NOT BE MOUNTED HIGHER THAN 9 1/2' AND MUST BE MOUNTED AT THE FINISHED CEILING LEVEL - WHETHER SOLID CEILING OR "DROPPED" CEILING.

EXTENDED CEILING KITS AVAILABLE

	PART NUMBER
11' Approx.	1000976-49
12 1/2' Approx.	1000976-63
14' Approx.	1000976-77

NOTE: WHEN PLACING YOUR ORDER, SPECIFY CORRECT ROD LENGTH AS SHOWN ABOVE. BE SURE TO TELL THE PERSON TAKING YOUR ORDER WHICH CEILING ROD LENGTH YOU NEED. FEEL FREE TO ASK QUESTIONS IF YOU ARE NOT SURE OR DO NOT UNDERSTAND THE MOUNTING.


**MEDICAL
ILLUMINATION**
INTERNATIONAL INC.

WHERE ILLUMINATION IS DEFINED

INSTRUCTIONS FOR DETERMINING CEILING ROD LENGTH WHEN ORDERING ANY CEILING MOUNT LIGHT:


SPECIAL NOTE: SEE THE MOUNTING INSTRUCTIONS FOR THE SPECIFIED MODEL TO DETERMINE THE WEIGHT AND MOVEMENT CAPABILITIES NECESSARY FOR THE SUPPORT.

**MEASURE THE DISTANCE FROM
THE FLOOR TO THE MOUNTING
SURFACE.**

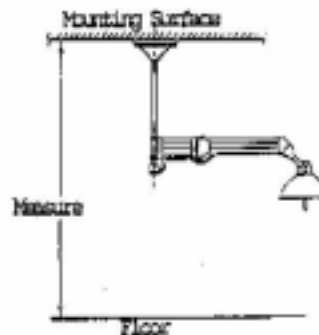


**IF FACILITY HAS AN ACOUSTIC TILE
TYPE "DROPPED CEILING", LIGHT CAN
BE MOUNTED ONE OF TWO WAYS:**

- A.** If the distance between the solid mounting surface and the dropped ceiling is not excessive and the distance from the mounting surface to the floor does not exceed 12' the light may be mounted as shown. Consult the rod length chart to determine the correct length required.

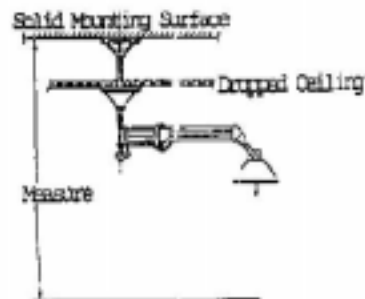

- B.** If light is to be mounted directly at the "dropped ceiling" level, then supports must be constructed above the acoustic tile ceiling - up to the solid surface above, to support the mounting.

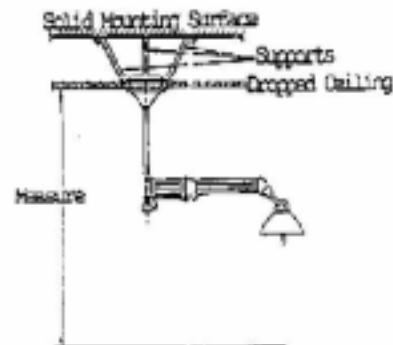

FOR QUESTIONS CONCERNING MOUNTING,
CALL THE CUSTOMER SERVICE DEPARTMENT
AT (818) 838-3025